

## **Life Pulse HFV with the VIP Gold/Sterling Ventilators**

Place in SIMV TCPL

Disconnect Flow sensor from ventilator and patient wye

Press Reset \*

Set appropriate PIP, PEEP, and Rate Parameters

\* Occasionally, on the VIP Gold, removing the sensor and pressing reset does not silence the alarm. If you turn the machine off and back on, that usually takes care of the problem.

## **Life Pulse HFV with the VIP Ventilator and Partner Monitor**

Place in SIMV TCPL

Turn Assist Sensitivity OFF

Turn Termination Sensitivity OFF

Turn Pressure Support OFF

Disconnect Flow Sensor and unplug from Partner Monitor

Press Reset

Set appropriate PIP, PEEP, and Rate Parameters