

## **Operating the Life Pulse HFV in Tandem with the Dräger Evita Ventilator**

With background IMV:

1. Use the PCV+ mode
2. Select desired P<sub>insp</sub> pressure setting (absolute pressure) Set PIP setting and PEEP settings
3. Dial out the pressure support (i.e., set to 0)
4. Set the respiratory rate (the apnea alarm can be adjusted to 60 seconds or turned off if needed)
5. Turn off the neoflow sensor and take the sensor out of line.
6. Set the Slope to avoid pressure overshoot.
7. Set Inspiratory Time appropriate to promote alveolar recruitment (sigh breath)

With very small background IMV: This is basically the same as CPAP

1. Use the CPAP / PS mode
2. Select the PEEP setting
3. Dial out the pressure support (i.e., set to 0)