



# CONNECTION INSTRUCTIONS FOR GENOSYL® DS AND BUNNELL HFJV FOR SERVO PRESSURES <7PSI

1. Procure one (1) GENOSYL DS Bunnell Tee Adapters (Vero PN 602532), described in **Figure 1**.

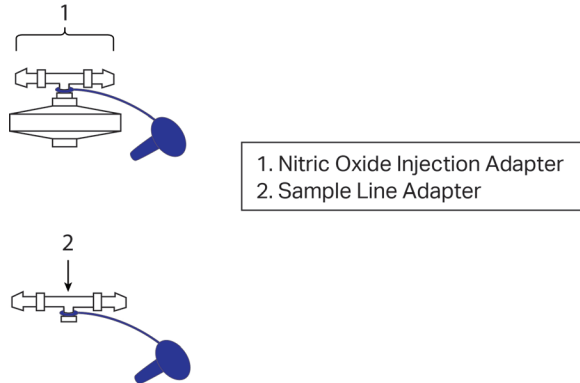


**Figure 1:** GENOSYL DS Bunnell Tee Adapter

2. Make one cut in the middle of the Gas Outlet Hose and another cut in the Bunnell Delivery Tubing two inches from the LifePort Adapter as shown in **Figure 2**.

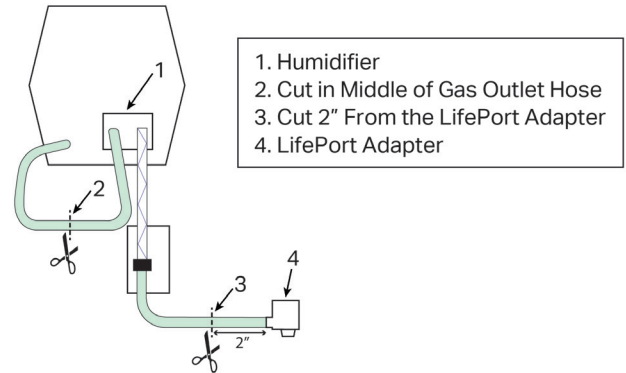

**Figure 2:** Bunnell Gas Line Modification

3. Insert the Nitric Oxide Injection Adapter (with the filter) at the cut in the center of the Gas Outlet Hose and the Sample Line Adapter (without the filter) at the cut in the center of the Bunnell Delivery Tubing, as shown in **Figure 3**.

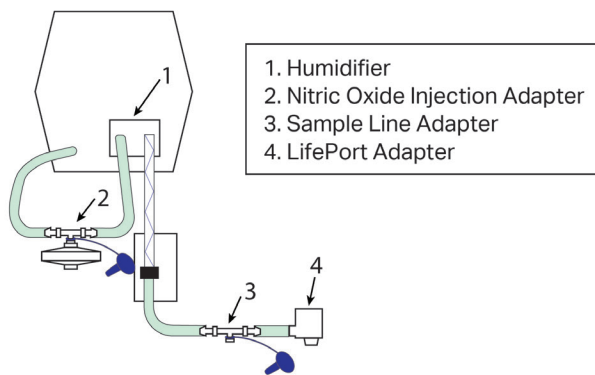

**Figure 3:** GENOSYL DS Bunnell Tee Adapters

4. Connect the red NO Injection line to the Nitric Oxide Injection Adapter and connect the blue stopcock on the Gas Sampling line to the Sample Line Adapter. Reference GENOSYL DS Operator's Manual System Set-Up and Connections for additional details.

For technical support, contact the Partnership365™ Care Team at **1.877.337.4118**