

## Closed Suction System Options

**AirLife** (provided by CareFusion, 800.231.2466) this system works well with the Bunnell LifePort adapter. The *AirLife*™ Closed Suction System features the Verso™ adapter, the most versatile adapter for safe and simple access to your ventilated patient's airway.

<u>Cat. No.</u>	<u>French Size</u>
50-3505	5 Fr
50-3506	6 Fr
50-3507	8 Fr

[Click for link to AirLife webpage](#)

One advantage of the AirLife Verso adapter is the guide post inside the 15 mm connector that directs the suction catheter into the ET tube lumen. With the Ballard product the catheter can turn as it is being advanced and catch inside the LifePort adapter.

**Ballard** (now Kimberly Clark , [www.kchealthcare.com](http://www.kchealthcare.com) , 800-524-3577) Makes the Neonatal Elbow model of their Trach Care product that works with the Bunnell LifePort adapter. It comes in various sizes:

<u>Cat. No.</u>	<u>French Size</u>
202	5 Fr
206	6 Fr
208	8 Fr
210	10 Fr

**B&B Medical Technologies** ([www.BandB-Medical.com](http://www.BandB-Medical.com) , 800-242-8778

Bodai NEO<sub>2</sub> – Safe

This is a circuit “Y” connector with a duckbill valve built into it, so you can suction without losing pressure in the circuit.